Smaronia

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Smaronia

Gel for atrophic vaginitis and vaginal dryness

Smaronia™ is a gel specifically formulated to be suitable for vaginal treatment. The components of the gel contribute to restore the vaginal physiological conditions and functions. Smaronia™ has soothing, lubricant and moisturizing actions, contributes to the local trophism and to maintain or restore the integrity of the mucosal tissue. The gel is mucoadhesive. The film barrier formed on the vaginal mucosae contributes to repair lesions and to restore the physiological properties of tissue.

Indication	Warnings
Smaronia™ is indicated in the treatment of vaginal symptoms in genitourinary syndrome in Menopause (GSM). Smaronia™ contributes to reduction of vaginal dryness and vaginal atrophy. Smaronia™ restores the vaginal trophism and the physiological function of vaginal mucosae.	Disposable applicator. Do not re-use once open. A transient burning may occur with the first applications that will disappear with regular use. If irritation persists, consult a healthcare professional. Keep out of reach of children. Smaronia™ does not contain hormones. Do not use if the primary pack is damaged. Compatibility with other medicaments has not been investigated. Do not use at age under 18.
Contraindication and Interaction	Clinically tested.
Known allergy for any of the ingredients. No known interaction with other medication.	Manufactured by NMTECH Italia Srl Via Vittorio Veneto 7 00187 Rome – Italy
Direction	Licensed by LINNEA SA Via Cantonale CH 6595 Riazzino – Switzerland
Intravaginal application. Wash hands thoroughly before applying. Remove the cap from the applicator. Use preferably in supine or lying position.	Distributed by FLOREALIS Grandagardur 16 101 Reykjavik – Iceland
Insert the content of one applicator into the vagina. Squeeze the applicator until it is completely empty. Do not wash the area after application. Smaronia™ Vaginal Gel can be used daily.	
Vaginal dryness (mild and moderate): Apply the gel daily, preferably in the evening, for a minimum of 7 consecutive days. In case of recurrence, repeat the treatment cycle. Apply every three days or according to needs.	
Vaginal symptoms in genitourinary syndrome in menopause (GSM) and/or vaginal dryness (severe) with vaginal atrophy: Apply the gel daily, preferably in the evening, consecutively for 21 days. To maintain benefits, repeat the treatment and apply preferably every three days or according to needs.	

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