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Adjuvant medical device used in the prevention of herpes simplex 2 (HSV-2) recurrence, in the treatment of bacterial, fungal and viral infections.

Use: The medical device Lijonía™ fast is a quick dissolving softgel formulation containing several different components, which, based on their properties, perform a protective barrier function as an adjuvant for the treatment of bacterial, fungal and viral infections, thereby forming a protective barrier, to isolate and protect tissue from external irritating factors, and help maintain and restore normal physiological conditions.

The medical device provides a dual action: Isolates the application area from external microbial aggressions; Limits the microbial colonization in the area of application, creating the best conditions for restoring the natural physiological conditions of tissue.

Instructions for use: Before use, wash the genital area thoroughly. Insert the softgel capsule deeply into the vagina. Repeat the application once daily for seven days. The medical device should be applied when first symptoms appear (itching, burning, discomfort etc.) before they occur as active lesions. No period of suspension is necessary following topical treatment.

Warnings: Only use the device with an intact epithelium. Do not use the product for any applications other than those stated in the package insert leaflet. Wash your hands thoroughly before any application. Do not use the device after the expiry date which is stated on the carton. Store at room temperature, away from heat sources. Do not freeze. Keep out of the reach of children. For topical use - do not swallow. Do not use if package is found open or damaged. Do not sterilize. Do not use in case of known hypersensitivity to any of the components. Discontinue treatment in case of abnormal skin reactions. In case of prolonged use, consult a doctor. The device is neither a spermicide nor a contraceptive.

Contraindications: There are no known phenomena of overdose. Known hypersensitivity to any of the components. There are no known interactions with drugs and medicinal substances.

Composition: TIAB (titanium dioxide microcrystals with covalently bonded silver ions), sodium hyaluronate, *Aloe Barbadensis* extract, caprylic/capric triglyceride, PEG-6 stearate (and) glycol stearate (and) PEG-32 stearate, glyceryl stearate.

Storage conditions: Keep away from humidity and heat sources. If exposed to sunlight, the product might take a slightly yellowish colour, which will not affect the functionality and shelf-life of the product.

Manufactured by
NMTECH Italia Srl
Via Vittorio Veneto 7
00187 Rome – Italy

Licensed by
LINNEA SA
Via Cantonale
CH 6595 Riazzino – Switzerland

Distributed by
FLOREALIS
Grandagardur 16
101 Reykjavík – Iceland